# TRIPSTONE STRATONE

M - 70 - 15 - 0862 (04) / 22 - Feb - 2021

**Instructions for use**  
ENGLISH

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**Promedon**

**INSTRUCTIONS FOR USE****DESCRIPTION**

The universal Nitinol stone retrieval baskets STRATONE (Ref.: 3500-0215-EP; Ref.: 3500-0215-EPP; Ref.: 3500-0215-EPP3 and Ref.: 4x0090-12/090/3.0) / grasper TRIPSTONE (Ref.: 3x22.8-12-15-GE/090-3) are comprised of three main components:

1. Stone retrieval basket/grasper.
2. Sheath.
3. Detachable handle.

**PACKAGING CONTENTS**

STRATONE/TRIPSTONE are comprised of:

- 1 (one) stone retrieval basket (STRATONE) / grasper (TRIPSTONE).

**INDICATIONS FOR USE**

STRATONE and TRIPSTONE are indicated for the endoscopic removal of urological stones.

**CONTRAINDICATIONS**

- They have not been described.

**PRECAUTIONS**

- Some urological stones may be too large for this type of procedure.
- Promedon did not design these devices to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- Single-use sterile product (sterilized with ethylene oxide). Any institution, practitioner or third party who reprocesses, refurbishes, remanufactures, sterilizes, and/or reuses these disposable devices must bear full responsibility for their safety and effectiveness.
- Operating room conditions must be in accordance with local hospital, administrative or government procedure.
- After use, dispose of the product and packaging. It is recommended to perform in accordance with local hospital, administrative or government procedure.
- Do not use the device if the packaging (sterile barrier) is open or damaged.
- Only a physician familiar with the endoscopic urological stone elimination procedure is qualified to use this device.
- This device must not be in contact with any electric instruments.

**STORAGE**

It is advisable to store the device in its original packaging at: Room temperature.

Use before the expiry date specified on the packaging.

**POTENTIAL COMPLICATIONS**

Endoscopic urological stone elimination procedures should only be performed by physicians familiar with the inherent potential complications of this type of procedures. Complications may occur during or after the procedure, and may include the following:

- Edema.
- Dissection.
- Urosepsis.
- Perforation of the ureter.
- Iatrogenic infections.

**PROCEDURE****PREPARATION:**

- Make sure the product packaging is not damaged. Do not use the product to perform the procedure if the packaging is damaged.
- Before the procedure, remove the product from its packaging in a suitable area, taking the necessary precautions for handling sterile products.

- Introduce the optical system into the urinary tract.
- Maneuver the closed retrieval basket (STRATONE) through the optical system towards the stone, so that the basket end basket is positioned right over the stone. In the case of TRIPSTONE the procedure is similar, but grasper must be inserted in front of the stone.
- Trigger the handle control in order to open the retrieval basket (STRATONE) / grasper (TRIPSTONE).
- Maneuver the device to trap the stone in the retrieval basket (STRATONE) / grasper (TRIPSTONE).
- When the stone is in the retrieval basket (STRATONE) / grasper (TRIPSTONE), slide the handle control back to close the retrieval basket (STRATONE) / grasper (TRIPSTONE) in order to trap the stone.
- Remove the trapped stones.

NOTE: After removing the handle, it is safe to remove the sheath through the optical system, while the retrieval basket remains in place. See Handle removal.

**HANDLE REMOVAL:**

STRATONE (Ref.: 3500-0215-EP; Ref.: 3500-0215-EPP; Ref.: 3500-0215-EPP3): Unscrew the handle and remove the sheath by loosening the Luer connector (red) attached to the handle.

STRATONE (Ref.: 4x0090-12/090/3.0) and TRIPSTONE (Ref.: 3x22.8-12-15-GE/090-3): Press the button on the back of the device and remove the lever from the handle. Using the distal end of the handle, turn the lever screw in the direction opposite to that indicated by the arrows. Remove the sheath.

**SYMBOLS USED IN LABELS**

	PRODUCT CODE
	LOT NUMBER
	SERIAL NUMBER
	CAUTION
	EXPIRY DATE
	DO NOT REUSE
	DATE OF MANUFACTURE
	STERILE. STERILIZATION METHOD: ETHYLENE OXIDE
	MANUFACTURER
	READ THE INSTRUCTIONS FOR USE
	DO NOT USE IF PACKAGING IS DAMAGED
	DO NOT RESTERILIZE
	KEEP AWAY FROM SUNLIGHT
	KEEP DRY

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La cesta STRATONE (Ref.: 3500-0215-EP; Ref.: 3500-0215-EPP; Ref.: 3500-0215-EPP3 y Ref.: 4x0090-12/090/3.0) / pinza TRIPSTONE (Ref.: 3x22.8-12-15-GE/090-3) colectora de cálculos universal confeccionada con Nitinol consta de tres componentes principales:

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**PROMEDON S.A.**