# TUBE

## Malleable Penile Prosthesis

### **PENILE PROSTHESIS**

MALLEABLE - STERILE - SINGLE USE

**Patient Guide - ENGLISH**

### **PROTHESE PENIENNE**

MALLEABLE - STERILE - USAGE UNIQUE

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### **FORMBARE PENISPROTHESE**

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## PATIENT GUIDE

### DESCRIPTION

The TUBE Penile Prosthesis has been designed to be implanted into the penis for the management of impotence. It serves as a stiffener that helps a patient suffering from erectile impotence to achieve vaginal penetration (intercourse). It is inserted into the corpora cavernosa through surgery. The TUBE Penile Prosthesis consists of two hemiprosthesis. The twisted and welded silver wires placed in the nucleuses are surrounded by a teflon which TUBE contains a medical grade silicone elastomer cover.

For vaginal penetration (intercourse) the prosthesis should be placed in erect position by hand. However, it is also possible to place the penis in relaxed position, due to its excellent placing memory that is given by the implant nucleous.

Prosthesis in Erect Position

Prosthesis in Down Position

### PROSTHESIS USE

Most patients are able to use their prosthesis six to eight weeks after their surgery. But some patients require a little more time. It depends upon your health and how fast you heal. Check with your doctor before first using your prosthesis or for any question you have about it. It should bend easily into the erect position or down for concealment.

Here are some tips to follow:

- It is recommended that partners become involved with the patient's recovery and his new prosthesis. Some time is needed to adjust to the prosthesis in the physical relationship.
- If you could ejaculate and achieve orgasm before surgery, you should be able to do so afterward. If they were not possible due to a physical or organic condition, the implant probably will not restore these abilities.
- Prosthesis use should not cause discomfort for your partner. You may feel some apprehension or even mild discomfort for a while. Lubrication may relieve the discomfort that couples might experience, who have not had sexual intercourse for some time.
- No especial care is needed for an implant. AVOID the following conditions that could occur during some recreational or sexual activities: repeatedly bending a prosthesis up and down, subjecting the prosthesis to repeated vibrations or being hit by a strong blow to the groin. An implant exposed to these conditions could lose its rigidity or break.

### MAGNETIC RESONANCE (MR) ENVIRONMENT

Non-clinical testing performed in the worst-case product has demonstrated that Tube Malleable Penile prosthesis implants are MR conditional. A patient with these devices can be safely scanned in an MR system meeting the following conditions:

- Static magnetic field of 1.5 Tesla and 3 Tesla, **with**
- Maximum spatial field gradient of 12,800 G/cm (128 T/m)
- **Maximum force product of 231 T<sup>2</sup>/m**
- **Theoretically estimated maximum whole body averaged (WBA) specific absorption rate (SAR) of 2 W/kg (Normal Operating Mode)**

Under the scan conditions defined above, the Tube Malleable Penile prosthesis implants are expected to produce a maximum temperature rise of less than 1.5°C (2 W/kg, 1.5 Tesla) RF-related temperature increase with a background temperature increase of ≈ 1.3°C (2 W/kg, 1.5 Tesla) 0.6°C (2 W/kg, 3 Tesla) RF-related temperature increase with a background temperature increase of ≈ 0.6°C (2 W/kg, 3 Tesla) after 15 minutes of continuous scanning.

In non-clinical testing, the image artifact caused by the device extends approximately 3.74 mm from the Tube Malleable Penile prosthesis implants when imaged with a spin echo pulse sequence and a 3 Tesla MR system.

### WARNING

The patient should consult his physician if he:

- Has difficulties when urinating.
- Feels pain in the gland or the scrotum immediately after surgery,
- Sees that the prosthesis finds it difficult to keep penis either in erect or down positions.
- Feels discomfort, throbbing of heart or prosthesis movement.

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PROMEDON S.A.