


# TUBE

| Malleable Penile Prosthesis |

## Surgical Technique

### QUICK GUIDE

1

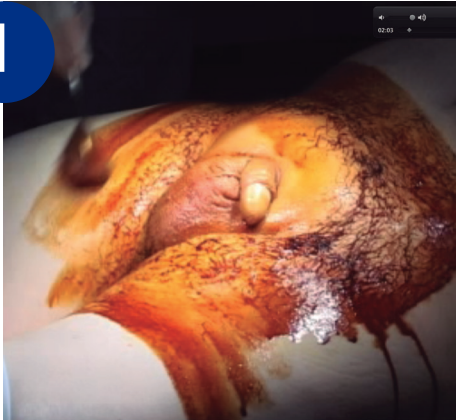

*Shave and sterilize the area for 5 min.*

2



*Isolate the perineum area of the incision using a U drape.*

3



*Introduce a Foley catheter to better identify the urethra.*

4



*Median longitudinal incision of approximately 5 cm in the union of the penis and the scrotum. The incision should be quite superficial, to preserve the urethra.*


5



*Bilateral dissection towards the lateral side of the Corpora Cavernosa (the Buck Fascia, the Dartos Fascia and the Tunica Albuginea)*



6

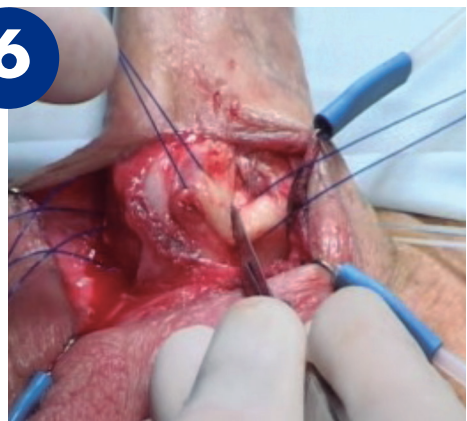


*Incision of the Corpora Cavernosa: two points of repair, 2-3 cm. in length, 1 cm. lateral to the urethra.*

# Surgical Technique


## QUICK GUIDE

7

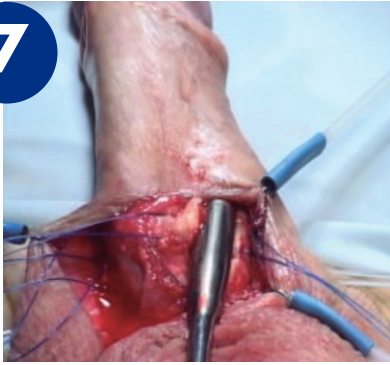


*Dilatation of the Corpora Cavernosa Distal: the person who performs the dilatation must also control the glans. Always slightly in a lateral direction to avoid cross-over perforation. Distal to the base of the glans. Dilatation up to 13 if possible.*

8

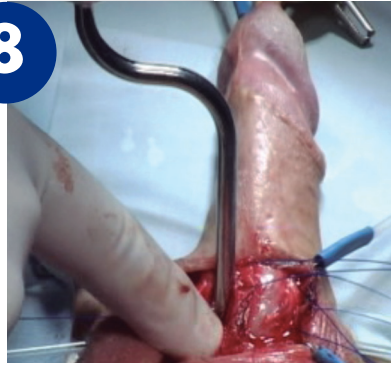


*Dilatation of the Corpora Cavernosa Proximal: always in a slightly lateral direction. The septum division is thinner than in the Distal. Until feeling contact with the ischiopubic ramus.*

9

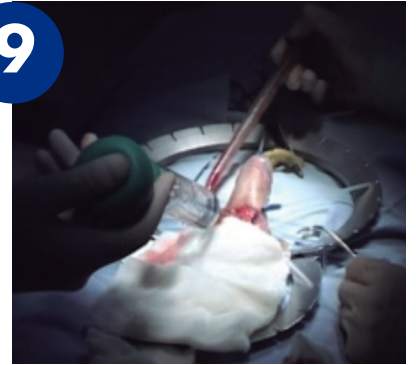


*Wash the Corpora Cavernosa with Gentamicin solution (80 mg / 500 ml saline solution)*

10

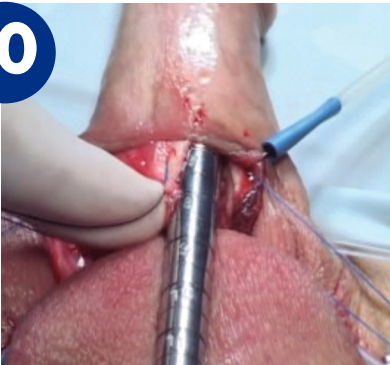



*Measure the Distal Length: measure to the point to repair the Tunica Albuginea.*

11

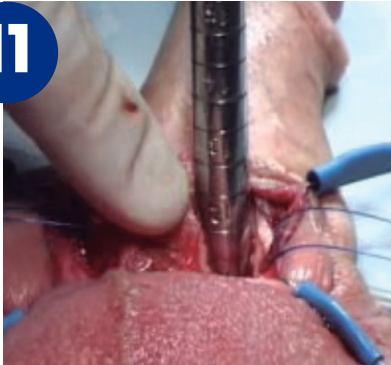


*Measure the Proximal Length: measure to the point of the last visible number.*

12

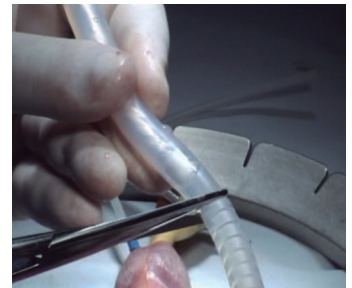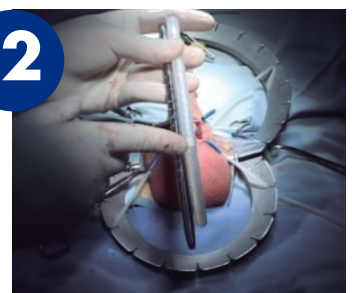

*Choice of length: according to the useful or functional length of the prosthesis. The proximal part should be cut as much as possible. If using Rear Tips, maintain a minimum of two segment to connect the Rear Tips or Extenders.*

13

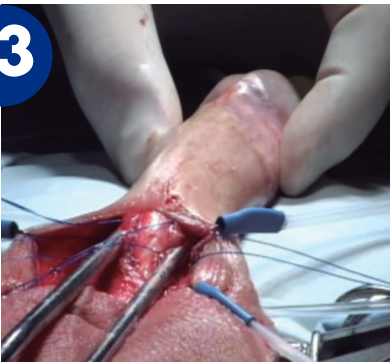

*Choice of width: use the widest diameter possible for the penis, maintaining aesthetic harmony.*

14



*Close the incision: suture PDS\*II 3-0.*