

DEACTIVATION PROCEDURE

REQUIRED MATERIALS:

- Sterile 10ml Syringe.
- Non-coring needle 25G x 5/8".

OTHER RECOMMENDED NON-CORING NEEDLES:

- Gripper Micro® Blunt Cannula, Non coring safety needle 20G x 1"(Ref: 21-3257-24).
- Surecan® prosty 20G x 40mm (Ref: 04439953).
- Surecan® Straight non-coring needle 22G x 30mm (Ref: 04439848).

STEPS

If the patient needs catheterization or transurethral procedure for any medical intervention, VICTO should be deactivated by removing liquid from the system as follows:

- 1** Identify the control pump in the patient's scrotum. Scrotal skin should be cleaned with antiseptic substance to minimize risk of infection.

NOTE Local anaesthesia around the area may be applied in accordance with hospital approved procedure, taking care of not damaging the control pump during this process.

- 2** Using clean gloves, palpate the control pump through the scrotal skin and locate the access port. (Fig.1)

NOTE You should identify the round port of the pump beneath the skin and hold it firmly in a fixed position. (Fig.2)


- 3** Connect an empty 10ml syringe to a sterilized non-coring needle.

- 4** Inject the non-coring needle into the access port until it contacts the titanium protection cup. The insertion of the needle should be perpendicular and concentric to the port. (Fig.3)

WARNING Do not use a different needle than the indicated ones because it may damage the self-sealing port affecting the implant's functionality. Precautions should be taken when inserting the needle, so as not to puncture the walls of the pump other than the self-sealing port.

- 5** Extract 7ml from the VICTO by pulling the plunger of the syringe. (Fig. 4)

- 6** Remove the needle from the port of the pump.

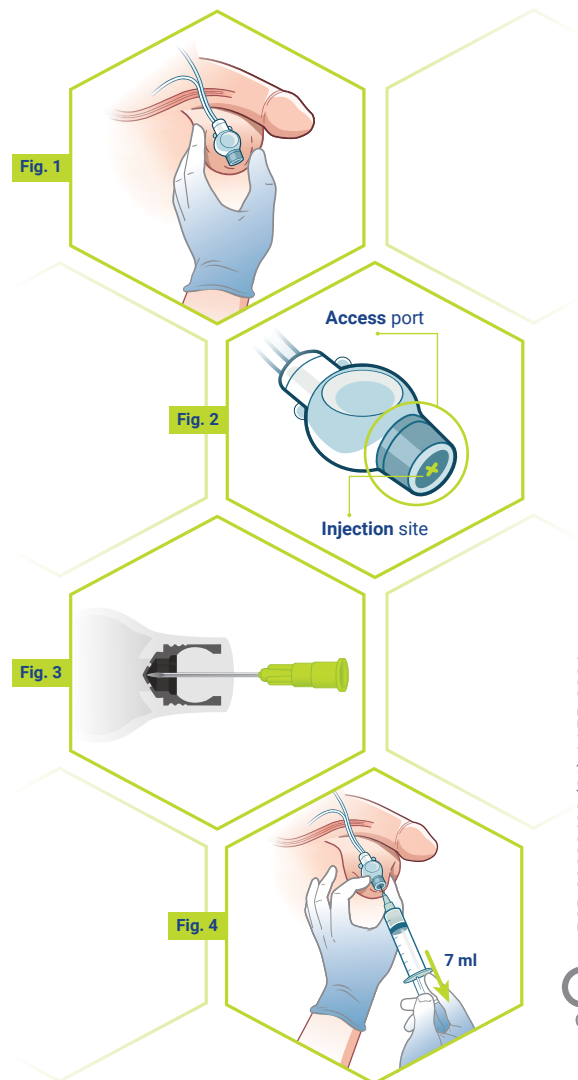

PSD-00000492 (01) / APR-2024



IMPORTANT: After performing the deactivation process, the VICTO sphincter will no longer provide pressure around the urethra. To restore its normal functioning after the patient's medical intervention has been completed, 7ml should be re-injected into the VICTO system. Please contact the surgeon indicated on the device identification card (VICTO ID card) to re-activate the product.