

## ORDERING INFORMATION

### Starter Kits

DESCRIPTION	CUFF SIZE	CODE
VICTO STARTER KIT 3.7	3.7	KIT-VICTO-37
VICTO STARTER KIT 4.0	4.0	KIT-VICTO-40
VICTO STARTER KIT 4.5	4.5	KIT-VICTO-45
VICTO STARTER KIT 5.0	5.0	KIT-VICTO-50
VICTO STARTER KIT 5.5	5.5	KIT-VICTO-55

### Accessories Kit

DESCRIPTION	CODE
VICTO ACCESSORIES KIT	KIT-AC-02

[victosphincter.com](http://victosphincter.com)

### Magnetic Resonance (MR) Environment

Non-clinical testing has demonstrated that Victo is MR conditional.  
A patient with this device can be scanned safely under the following conditions:



- Static magnetic field of 1.5 Tesla and 3 Tesla, with
- Maximum spatial field gradient of 12,800 G/cm (128 T/m)
- Maximum force product of 231 T<sup>2</sup>/m
- Theoretically estimated maximum whole body averaged (WBA) specific absorption rate (SAR ) of < 2 W/kg (Normal Operating Mode)

**Promedon**  
Urology and Pelvic Floor



## TAKING THE ARTIFICIAL URINARY SPHINCTER CONCEPT TO THE NEXT LEVEL

- Minimally invasive adjustment procedure
- Pre-connected device
- Ergonomic pump design




# VICTO ARTIFICIAL URINARY SPHINCTER.

VICTO is a preconnected artificial urinary sphincter that enables pressure adjustments with a minimally invasive ambulant procedure.


## MINIMALLY INVASIVE ADJUSTMENT PROCEDURE

With its unique pump design, the system pressure in VICTO can easily be adjusted through the self-sealing port, in an outpatient and easy-to-perform procedure.

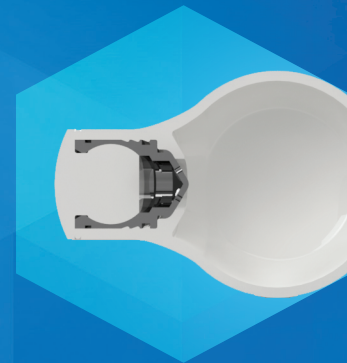


## ERGONOMIC PUMP DESIGN

The soft silicone pump has an easy-to-palpate design, which makes it simple for the patient to control. With only one pump stroke the urethra is decompressed and micturition is enabled.

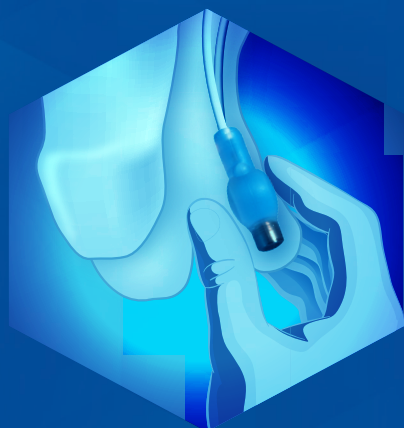

## PRE-CONNECTED DEVICE

VICTO is provided pre-connected, ready for filling and implantation.

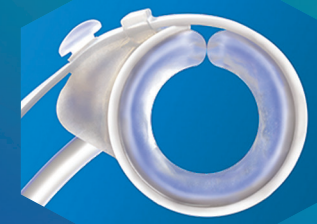

VICTO is available in 5 different cuff sizes, which can be selected intraoperatively using the cuff size selection tape, provided in the Accessories kit.

3.7<sub>CM</sub>

4.0<sub>CM</sub>

4.5<sub>CM</sub>

5.0<sub>CM</sub>


5.5<sub>CM</sub>

## ACTIVATION & ADJUSTMENT PROCEDURE

Preparation during surgery: +13ml

Activation +4ml

Individual adjustments(+1 ml), if needed

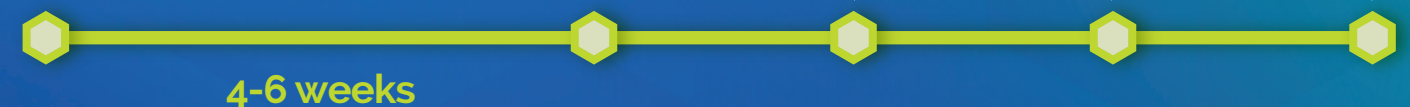

## NEEDLE RECOMMENDATION:

The usage of a non-coring needle 25G is recommended for the filling or liquid removal procedures performed during the Victo activation, adjustment or deactivation.

## INTENDED USE

VICTO is intended for the reestablishment of urinary continence in the defined patient population.

## INDICATIONS

VICTO is indicated for the treatment of male stress urinary incontinence.

## CONTRAINDICATIONS

- Uncontrolled detrusor overactivity
- Untreated or irreversible infravesical obstruction
- Impaired cognitive or manual abilities which restrict the patient to handle the device
- Untreated acute urinary tract infection or active infection at the surgical sites (i.e. Perineum, scrotum, lower abdomen)
- Known allergy to silicone.